900 2917

CYBERTECH CT CLEAN TRAY CLEANER

CT Clean Tray cleaner is a concentrated solution for cleaning impression trays and dental instruments.

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It provides a fast and effective means for removing traces of alginates, zinc oxide eugenol, carboxylate and phosphate cement, as well as thermoplastic impression compounds and waxes. The ingredients are biodegradable. **Directions for use:** CT clean Tray cleaner should be used in a concentration of 5-10%. It can also be applied in undiluted form in order to remove slubbery residues. Add the contents of the dosage container to 1 litre of warm water, immerse the soiled objects in the cleaning solution for 45 minutes, depending on the degree of soiling. The clearing solution can also be used in an ultrasound bath. CT clean Tray cleaner is suitable for removing traces of alginates that have not been completely oxidized. **Precautions:** Risk of corrosion! 648/2004/EG: Entre 5% and 15% tensioactivos surfactantes, between 15% and 30% Na-EDTA, perfumes. **Danger:** Causes serious eye damage. Causes skin irritation. Flammable liquid and vapour. Keep away from heat/spark/open flames/hot surfaces. -No smoking. Keep container tightly closed. Wear protective gloves and eye protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. Call a POISON CENTER or doctor/physician. If you feel unwell. If skin irritation occurs: Get medical advice / attention. In case of fire: Use low pressure water spray. Extinguish by spraying on the walls. Do not breathe vapour. Do not get inside the container. Dispose of contents contained in accordance with regional/national/international regulations. Contains: Sulfuric acid, C 13-17-sec-alkane, sodium salt.

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DE Healthcare Ltd
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
E-Mail: info@cybertechbrand.com
www.cybertechbrand.com

Rev.2015/04