**Applicable for the following film processors:**  
**DÜRR** (AC 245, XR-24, XR 24II, XR 24PRO, XR25, XR24easy)  
**PHILIPS** (Clarimat Super S2, S3)  
**GIRARDELLI** (X-20, X-24)  
**VELOPEX** (DE260, Freedom X-Ray, Velopex extra X)  
**GENDEX** (GX 300)

**Preparation instructions for automatic film processors:**  
 1. Preparation of the baths — see illustrations.  
 2. Regeneration and utilization of the baths:

**2.1 Automatic Regeneration:**

For machines with automatic regeneration, the baths must be renewed every 3 months. More often with very low film throughput, as the oxidation of the developer cannot be completely balanced due to the fewer regeneration dosages. Never empty the regeneration containers completely, otherwise air will be pumped into the baths, which may lead to premature oxidation of the solution.

**2.2 Manual Regeneration:**

Approximately 7 dental panoramic films can be processed a day in one machine (approximately 2.5 litres) with regular usage by manual regeneration. Add twice a week 0.5 litre of the remaining solution from the corresponding cans to the developer and fixer baths. These regeneration quantities apply to the output of 7 panorama films a day. With increased output (specialized practice), regeneration will be required more often. In such cases, an automatic regenerator is highly recommended. Using this, the baths have to be renewed every 3 weeks.

**Cleaning of the machine:**

Processors has to be thoroughly cleaned every 3 months. Please use current cleaning sets for cleaning the processors

**Important Notice:**

Please take care that cans and caps of the developing and fixing solutions should not be exchanged under any circumstances. Even slight contamination will impair the results. Before mixing the solutions, the cans must always be rinsed thoroughly with water. Please take care that, after mixing, at first fixing and afterwards developing solution should be filled carefully into the corresponding tanks. Please keep strictly to this order, as the slightest contamination will render the solution unusable.

**CAUTION:** Irritant. Irritating to eyes and skin. In the event of contact with eyes, rinse them thoroughly with water and consult a doctor. Please wear protective gloves when handling the concentrate.

**Recycling instructions:**

Rinse empty concentrate bottles with water twice and supply to the Dual System.

**900 3007**

**CYBER TECH**

**XRAY AUTOMAT SET**

**Special X-Ray Set for Automatic Film Processors**  
**INSTRUCTIONS FOR USE**

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CE REP  
 Authorised representative:  
**DE Healthcare Products**  
 Gillingham  
 ME8 0SB U.K.

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