Irritant
Irritante
Irritant
Reizend
Irritante

Toxic
Tóxico
Toxique
Giftig
Tossico

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CE 0473

DE Healthcare Products
Gillingham
ME8 0SB U.K.

1

2

3

4

5

6

7

8

9

Cybertech Fill Desensitising Gel

INSTRUCTIONS FOR USE

Description: Cybertech Fill Desensitising Gel is a unique, sustained release, 3% potassium nitrate and 0.11% weight by weight fluoride ion, sticky, viscous gel. Cybertech Fill Desensitising Gel is designed to be used in a custom-fabricated tray.

Indications: Cybertech Fill Desensitising Gel provides a film-like varnish for sensitive teeth, sealing dentinal tubules in exposed dentin and other areas where post-operative sensitivity is a concern. The product is used either by a dental professional in the office or provided to the patient for home treatment of dentin sensitivity.

Procedure:

- Fabricate tray to meet patient's needs. See laboratory instructions. Cybertech Fill Desensitising Gel may be applied in the same tray and manner as bleaching gel or into a bleaching tray if available.

Note: *If large retention areas exist from prior periodontal surgery that might present problematic impression removal from the teeth, block out embrasures with Block out material.*

- Cybertech Fill Desensitising Gel is dispensed in 1.2ml unit dose syringes. A complete arch should use most of the contents of a syringe. If sensitivity is localised, place gel in affected area(s) and a small amount in the molar region to hold tray in place.
- The length and number of times a tray is worn depends on condition, patient and clinician. Recommended treatment times can range from 15 minutes to 1 hour.
- Tray should be removed at meal times. Immediately following tray removal, patient should brush and rinse teeth with water to remove all remaining gel.
- DO NOT swallow solution during rinsing. Wash tray and store in a tray storage case. Fresh gel should be used with each application.
- Gel may be warmed before placing in tray to avoid thermal sensitivity.

Laboratory Instructions:

- Pour and trim cast as you would for a standard bleaching tray. Apply 0.5-1.0mm of block-out resin to a clean, dry cast on areas in which patient is experiencing sensitivity (see Fig. 1 & 2). Also apply resin to areas where there are open retention areas, etc. For periodontal patients, clinician may determine that pre-operative impressions are indicated. In such cases, reservoir spacing will be created post-op because of recontoured and apically positioned gingiva.

- Cure block out for about 2 minutes in light curing unit. A normal curing light can be used instead (according to instructions). Wipe off oxygen inhibition layer.
- Prepare a soft splint acc. to manufacturer's instructions.
- Trim excess tray material, allowing 4-6mm to overlap onto gingival tissue. Contour tray margin to avoid anatomical structures such as frenum attachments, bone growth, etc.
- Return tray to model; check tray extensions. Gently flame polish edges, one quadrant at a time.
- While still warm, hold periphery of each segment firmly against model for three seconds with water-moistened finger. If an area is short of the desired length, gently heat and push tray material to the desired location. If this thins the material too much, a new tray should be fabricated.

Precautions:

- Keep Cybertech desensitisation product away from small children.
- Not for use overnight.
- Do not use smoke or eat while using this product.
- Not intended for use by pregnant women.
- If any problems occur, discontinue treatment and notify your dentist.
- Keep out of heat and/or sunlight at all times. It should be stored at room temperature.
- Patients should not swallow large volumes of Cybertech Fill Desensitising Gel. Excess material at insertion and residue upon removal should be brushed off and rinsed: DO NOT swallow.

DENTIST INSTRUCTIONS & LAB INSTRUCTIONS

- Construct and dry the stone model, cutting palate and tongue areas out (Fig. 3).
- Vacuum adapt soft tray sheet (0.5 mm) to model. Cool and separate (Fig.4).
- For sensitive roots use LC block out resin on gingival margin to provide a reservoir space for Cybertech Fill Desensitising Gel. It is recommended to lap tray 2-3mm onto gingiva (Fig. 5).

General Precautions:

- For professional use only
- Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Confirm that patient has no known allergies to treatment materials.

PATIENT INSTRUCTIONS

- Follow your dentist's instructions on how to load gel into your custom tray. Use no more than 1/3 to 1/2

of syringe (Fig. 6).

- Brush teeth, then insert tray. Lightly tap tray to adapt tray sides to teeth. Remove tray after 15 minutes to 1 hour or as directed by your dentist (Fig. 7).
- Remove excess gel with clean finger or soft toothbrush. Rinse twice; do not swallow rinses (Fig. 8).
- Clean tray with soft brush and cool tap water. Store tray in case provided (Fig. 9).

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