900 6104

CYBERTECH

CT CLEAN BUR

For cleaning and disinfection of dental rotary instruments.

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Instructions for use: CT Clean Bur is an aldehyde free ready to use solution. Place the used instruments fully immersed into the solution for specified contact time. Rinse with water and dry thoroughly. Do not mix with other disinfectant or cleaners. Not suitable for Aluminium. Microbiological Effectiveness: Bactericidal according to VAH/DGHH, EN 13727, EN 14561. Fungicidal/Levurocidal (Candida albicans) according to VAH/DGHH, EN 13624, EN 14562. Tuberculoicidal according to VAH/DGHH. "Restricted viruses" (effective against membrane covered viruses such as HBV, HIV, HCV) according to BGA(RKI)/DVV. Contact Time: Bactericidal (high bio load) - 15min. Fungicidal/Levurocidal (high bio load) - 15min. TB (Tuberculosis)-Inactivation - 30min. Membrane covered viruses (i.e. HBV, HIV, HCV) with bio load - 15min. Ingredients: <5% non-ionic surfactants, perfumes.

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+H65890061040B

0473

DE Healthcare Ltd
(manufacturer)
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
Email: info@cybertechbrand.com
www.cybertechbrand.com

Rev.2015/04