900 6106

CYBERTECH

CT CLEAN CONCENTRATE ^{EVAC}

High performance solution designed for cleaning and disinfecting of dental suction system and mouth rinse basin.

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LOT

0473

 **DE Healthcare Ltd**
(manufacturer) Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
www.cybertechbrand.com

Rev.2015/04

Indications for use: CT Clean Evac concentrate must be first be diluted with water – to a 2 or 3% concentration. (20/30 ml concentrate with 1 ltr. water). Approx. 1 liter of solution should be sucked through. At normal usage the evacuation unit should be disinfected two to three times daily. Do not mix with other disinfectants or cleansers. The solution should not be left in the solution unit overnight. Microbiological Effectiveness: Bactericide in accordance to VAH/DGHH EN 13727, EN 14561 and EN 1276. Levurocide (Candida albicans) in accordance to VAH/DGHH and EN 13624, EN 14562, EN 1650. "Restricted viruses" (effective against Membrane covered viruses e.g. HBV, HIV, HCV) in accordance to BGA (RKI)/DVV. Efficacy: Bacteria and Candida albicans in accordance to VAH/DGHH instrument disinfection; 3,0% 2 min. Bacteria in accordance to EN 13727 and EN 14561 and EN 1276. Levurocide (Candida albicans) and Candida albicans in accordance to EN 1650 (high bio load); 2,0% 5 min. Bacteria in accordance to EN 13727 and EN 14561 (high bio load); 3,0% 2 min. Candida albicans in accordance to EN 13624 and EN 14562 (high bio load); 3,0% 2 min. Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA(RKI)/DVV and EN 14476 (high bio load); 2,0% 2 min. Safety instruction: Warning: Causes skin irritation, causes eye irritation. Wear protective gloves. Use with caution. Wash thoroughly after use. Do not touch your face. Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. IF ON SKIN: Wash with plenty of soap and water. Contains: N,N-Dimethyl-N-Diocytlammoniumchloride.

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