900 6107

CYBERTECH

CT CLEAN ^{EVAC} CONCENTRATE

High performance solution designed for cleaning and disinfecting of dental suction system and mouth rinse basin.

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5Le

0473

 **DE Healthcare Ltd**
(manufacturer)
**Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.**
Tel: +44 1634 266 056
www.cybertechbrand.com

Rev.2015/04

Indications for use: CT Clean Evac concentrate must be first diluted with water – to a 2 or 3 % concentration. (20/30 ml concentrate with 1 ltr. water). Approx. 1 liter of solution should be sucked through. At normal usage the evacuation unit should be disinfected one to two times daily. Do not mix with other disinfectants or cleanser. The solution should not be left in the suction device overnight. Microbiological Efficacy: Bactericide according to VAH/DGHM in EN 13272, EN 14561 and EN 1276. Levurocidio (Candida albicans) in accordance to VAH/DGHM and EN 13624, EN 14562, EN 1650. "Restricted viruses" (efficacy against Membran covered viruses e.g. HBV/HIV, HCV) in accordance to BGA (RKI)/DVV. Efficacy: Bacteria and Candida albicans in accordance to VAH/DGHM as Instrument disinfection (high bio load): 3.0% 5min. 2.0% 15min. Bacteria in accordance to EN 1276 (high bio load): 2.0% 5min. Candida albicans in accordance to EN 1650 (high bio load): 2.0% 5min. Bacteria in accordance to EN 13272 and EN 14561 (high bio load): 3.0% 2min. Candida albicans in accordance to EN 13624 and EN 14562 (high bio load): 3.0% 2min. Membrane covered viruses (i.e. HBV, HIV, HCV) in accordance to BGA (RKI)/DVV and EN 14746 (bio load): 2.0% 2min. Safety: Instructions: Warning: Causes skin irritation. Contains serious eye irritant. Wear protective gloves / eye protection. Wash thoroughly after handling. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. IF ON SKIN: Wash with plenty of soap and water. Contains: N,N-Dimethyl-N-Dioctylammoniumchloride.

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