CYBERTECH

CT CLEAN INSTRUMENT
POWDER

900 6108

For cleaning and disinfection of rotary and surgical Instruments
Para la limpieza y desinfección de instrumentos rotatorios y quirúrgicos
Nettoyant et désinfectant pour instruments chirurgicaux et rotatifs.
Zur Reinigung und Desinfektion von rotierenden & chirurgischen Instrumenten
Per pulire e disinfettare strumenti rotanti e chirurgici
Voor reiniging en desinfectie van roterende en chirurgische instrumenten

+H65890061080F

Danger
Peligro
Danger
Gefahr
Pericolo
Gevaar

2 kg e

see imprint
ver la fecha impresa
voir impression
siehe Aufdruck
vedi iscrizione
zie opdruk

LOT

(manufacturer)

DE Healthcare Ltd
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
www.cybertechbrand.com

Rev.2015/04

Instructions for use: Dilute the concentrated powder with water using the dosing spoon (15 ml = 12 g; 30 ml = 25 g) provided. Mix the powder with water as per contact time required and stir briefly. The powder will dissolve completely within 5 minutes. Place the instruments in the solution after use, hissing and air bubbles are covered by the solution. After disinfection rinse instruments thoroughly as one day. We recommend a final rinse with demineralized water. Do not mix with detergents or other disinfectants. Do not use on copper, bronze, chromed or nickel coated instruments with damaged surfaces. The solution should be changed after each clinical session or sooner if it is heavily contaminated (Using the concentration indicated, the instrument disinfectant cleaner can be used in an Ultrasonic bath. Final sterilization of instruments is recommended. **Microbiological effectiveness/Contact times:** Bactericidal according to EN 13727, EN 14561 (high load): 1% - 15min, 0.5% - 60min. Levurocidio (Candida albicans) according to EN 13624, EN 14562 (high load): 1% - 15min, 0.5% - 60min Td (Tuberculosis deactivation) according to EN 14348 (M. terrae): 3% - 60min. Enveloped viruses (e.g. HBV, HIV, HCV) pursuant to DVV/RIK and according to EN 14476 (BDV/Vaccinia with high load): 1% - 15min, 0.5% - 60min. **Danger:** Causes severe skin burns and eye damage. Wear protective gloves / eye protection. IP in EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with local/regional/national/international regulations. **Content:** didecylidimethylammonium chloride, N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine. Close the bucket immediately after removal of the powder.

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