900 6110

CYBERTECH

CT CLEAN INSTRUMENT
CONCENTRATE

For cleaning and disinfecting of rotating and surgical instruments.

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LOT

0473

DE Healthcare Ltd
(manufacturer)
**Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.**
Tel: +44 1634 266 056
www.cybertechbrand.com

Rev.2015/06

Instructions for use: Indications for use: CT Clean Instrument concentrate has to be diluted down with water. Take 980 ml water and 20 ml of the concentrate to prepare 1 l ready for use solution (2%). Heavy contaminated instruments should first be cleaned before submerging into the disinfectant. After disinfection, instruments are to be thoroughly rinsed with water and dried. Do not mix with other types of disinfectants or solutions should be cleaned. No rinsing should be performed in the same container. Rinse immediately if it is heavily contaminated (Using the concentration indicated, the instrument disinfectant cleaner can be used in an Ultrasonic bath of instrument). Microbiological Effectiveness: Membrane covered viruses (i.e. HBV, HIV, HCV), Bacteria (Candida albicans), tuberculosis, "restricted viruses" (effective against: Herpesviruses (i.e. HSV, HIV, HCV), Bacteria in accordance to VAH/DGHH, EN 13727, EN 14561 (high bio load): 3% 15min, 2% 30min, 1% 60min, Leucovoridin in accordance to VAH/DGHH and EN 13624, EN 14561 (high bio load): 3% 15min, 2% 30min, 1% 60min, TB (Inactivation secondo VAH/DGHH) in accordance to VAH/DGHH (alta carica biologica): 3% 15 min, 2% 30 min, 1% 60 min. Membrane covered viruses (i.e. HBV, HIV, HCV) 3% 15min, 2% 30min, 1% 60min. Ultrasonic bath (Bacteria and Candida albicans) in accordance to VAH/DGHH (high bio load): 3% 5min, 2% 10min, 1% 20min. Danger: Causes severe skin burns and eye damage. Wear protective gloves & eye protection. If IN CONTACT WITH WATER: Rinse thoroughly with water for several minutes until contact is broken. If present and easy to do, continue rinsing. If skin irritation occurs: Get medical advice/information. If eye irritation/irritation occurs: Get medical advice/information. If in CONTACT WITH MOUTH: Do NOT swallow. Do NOT use for mouth care. Take off immediately all contaminated clothing. Rinse skin with water (or shower). Call a POISON CENTER or doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. Contains: Didecylmethylpolyoxyethylammoniumpropionat, N-(3-Aminopropyl)-N-dodecylpropane-1,3-diamina.

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