900 6111

CYBERTECH

CT CLEAN INSTRUMENT
CONCENTRATE

For cleaning and disinfecting of rotating and surgical instruments.

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LOT

5Le

 DE Healthcare Ltd
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
www.cybertechbrand.com

Rev.2015/06

Instructions for use: Indications for use: CT Clean Instrument concentrate has to be diluted down with water. Take 980 ml water and 20 ml of the concentrate to prepare 1 l ready for use solution (2%). Heavy contaminated instruments should first be cleaned before submerging into the disinfectant. After disinfection, instruments are to be thoroughly rinsed with water and dried. Do not mix with other types of disinfectant or cleaners. The solution should be changed after each clinical session or sooner if it is heavily contaminated. Using the concentration indicated, the instrument disinfectant cleaner can be used in an Ultrasonic bath. Final sterilization of instruments is recommended. Microbiological Effectiveness: Bactericidal, leucoricid (Candida albicans), tuberculocidal, virucidal (enveloped viruses: HBV, HIV, HCV). Batterii e cellule vegetali: EN 13727, EN 14561 (high bio load): 3% 15min, 2% 30min, 1% 60min. Levorucidali in accordance to VAM/DGHM and EN 13824, EN 14562 (high bio load): 3% 15min, 2% 30min, 1% 60min. TB (Tuberculosis-inactivation in accordance to VAM/DGHM: 3% 60min. Membrana covered viruses (i.e. HBV, HIV, HCV): 3% 15min, 1% 30min. Ultrasonic bath (Batteri e Candida albicans in accordance to VAM/DGHM: (high bio load): 3% 5min. Danger: Causes severe skin burns and eye damage. Wear protective gloves / eye protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water for shower. Call a POISON CENTER or doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. Contains: Didecylmethylpolyoxyethylammonioipropionate, N-(3-amminopropil)-N-dodecylpropane-1,3-diamine.

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