900 6113

CYBERTECH

CT CLEAN SURFACE
DISINFECTION

Ready to use rapid surface disinfectant for noninvasive medical devices.

Desinfectante rápido, listo para usar, para el área médica y dental farmaci non invasivi.

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5Le 

DE Healthcare Ltd
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
www.cybertechbrand.com

0473

Rev.2015/04

Instructions for use: Spray the surface from a distance of 30 cm until completely covered. Allow to dry. Wipe residue with a disposable cloth. Use only on alcohol compatible materials (i.e. not suitable for acrylic glass). DIN 51755: approx. 25°C. Max. quantity 50ml per m². The total quantity per area is not allowed to be more than 100ml per m² of the ground area. Keep container tightly closed. Packing material should be completely emptied of contents before disposing. Microbiological Effectiveness: Battericida (incl. MRSA) according to VAH/DGHH, EN 1040, EN 1276; Fungicida/Levoricida (Candida albicans) according to VAH/DGHH, EN 1276, EN 1650; Tuberculocida according to VAH/DGHH, EN 14348; "Restricted Virucida" (effective against membrane covered viruses), i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV, Virucida against Adenovirus according to EN 14476; Contact Times: Battericida (incl. MRSA) according to VAH/DGHH, EN 1276 (high bio. load) - 1 min.; Fungicida/Levoricida (Candida albicans) according to VAH/DGHH, EN 1650 (i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV with bio load - 30sec. Adeno-Virus (EN 14476) - 30sec. 100g contains: 43,0g Ethanol, 8,0g l-Propanol, 0,04g Didecylidimethylammoniumchloride. Safety Instruction: WARNING: Flammable liquid and vapour. Keep away from flames/hot surfaces. No smoking. Wear eye-protection. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container in accordance with local/regional/national/international regulations.

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