CYBERTECH

900 6114

CT CLEAN WIPES CONTAINER MINT

Alcohol based wet wipes for rapid surface disinfection of non-invasive medical devices.

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www.cybertechbrand.com

Rev.2015/04

DE Healthcare Ltd
(manufacturer)
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
Tel: +44 1634 266 056
www.cybertechbrand.com

Instructions for use: Wipe surface thoroughly and allow to dry. Use only on material compatible with alcohol (i.e. Not suitable for material such as acrylic glass). Please note - the lifetime of the wipes in the dispenser tub is only 28 days after opening the retail pack. Microbiological Effectiveness: Bactericide (incl. MRSA) according to VAH/DGHH, EN 1040, EN 1276; Fungicide/Antifungal (Candida albicans) according to VAH/DGHH, EN 1650; Tuberculocidal according to VAH/DGHH, EN 14348; "Restricted Virus" (effective against membrane covered viruses such as HBV, HIV, HCV), Adeno Viruses, Contact Times: (with reference to the liquid): Bactericide according to VAH/DGHH, EN 1040 and EN 1276 (high bio load): 5 min (ac. - VAH); Candida albicans according to VAH/DGHH, EN 1276 and EN 1650 (high bio load) - 1 min TB (Tuberculosis)-inactivation according to VAH/DGHH, EN 14348 (high bio load) - 1 min. Adeno Viruses (EN 14478; high bio load) - 30 sec. Membrane covered viruses (i.e. HBV, HIV, HCV) according to BGA (RKI) DVV with bio load - 30 sec. Contents: 100g Solution, 100g solution contains: 43,0g Ethanol, 8,0g 1-Propanol, 0,04g Didecylmethylammoniumchloride, Safety Instruction: Flammable solid. Keep away from heat/spark/open flames/hot surfaces. No smoking. Wear protective gloves/protective clothing/eye protection/face protection. In case of fire: Use for extinguishing: CO2, powder or water spray. Store in a closed container. Flashpoint acc. to DIN 51755: approx. 25°C. Max. application: 50 ml per m². The total amount applied per room must not be more than 100 ml per square metre of floor space. Keep containers lightly closed. Listed by DGHH/VAH.

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