INSTRUCTIONS FOR USE

Radiopaque Glass Ionomer Luting Cement
Cyberlonoglass 1 chemically bonds to tooth structure & metal providing. Specially developed for long term restorations. Cyberlonoglass is designed for final cementation of metal inlays, onlays, posts and orthodontic brackets.

INDICATIONS:

- Base and liners
- Cementation of orthodontic bands
- Cementation of metal and porcelain (all types)
- Cementation of stainless steel crowns and orthodontics

DIRECTIONS FOR USE:

Powder/Liquid ratio (g/g) 2.4 / 1.0

Mixing time: 30 sec.

Working time: 2:30 – 3:00 min.

Final finishing commencing time (after seating the restoration): 4:30 min.

1. TOOTH PREPARATION:

Clean the cavity preparation with pumice and water. Rinse thoroughly and lightly dry and do not remove smear layer. Pulp capping with calcium hydroxide. Clean and dry inside of casting or stainless steel crown.

2. POWDER AND LIQUID:

Please mix powder + liquid ratio with ratio: 2.4g + 1.0g. This can be obtained by mixing of one level scoop of powder and three drops of liquid. Shake the bottle to loosen the powder. Overfill the spoon with the powder, level the spoon and place onto the mixing pad. Avoid compressing powder into the spoon with the inside wall of the bottle. Turn the liquid bottle vertically with the tip about 5cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense one drop. If any bubbles are present, lightly tap the bottle with the fingers holding it. Discard drops that are obviously not full sized. Use a small spatula to rapidly mix all the cement powder into the liquid. Usually one scoop of powder and 3 drops of liquid should provide for sufficient amount of mixed cement. The mixed cement should be thixotropic and have a smooth consistency and glossy appearance. Total mixing time is 30 secs. After use, tightly close both liquid and powder bottles to prevent exposure to moisture.

3. CEMENTATION TECHNIQUE:

Cementation of cast metal crowns or bridges: Coat internal surface of restoration with 1mm of cement and start to seat within 30secs of completing mix. Do not overfill. Note that higher temperatures shorten working time. Cementation of stainless steel crowns: Fill the adapted crown with cement and seat. To enable removal of orthodontic appliances, coat the occlusal surfaces of the teeth to be cemented with a fluoride gel or petroleum jelly prior to cementation.

Net setting time is 4 minutes after start of mixing. Remove excess cement at the first formation of gel stage. Do not use Cyberlonoglass with patients who show an allergy to the material. In case of allergic reactions immediately stop the application and advise the patient to consult a physician. If history of allergy to glass ionomer cements is known, do not use Cyberlonoglass.

CAUTION:

For Dental Use only.

Do not allow the cement mixture to contact the oral tissues or skin. In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water. In case of eye contact, immediately flush with water and seek medical treatment. Do not mix the powder or liquid of Cyberlonoglass with any other glass ionomer products. Do not use after expiry date.

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