INSTRUCTIONS FOR USE

CyberTemp Zircon

Dual cured permanent adhesive resin cement for Zirconia
CyberCyberZirconite * Dual cured permanent adhesive resin cement for Zirconia. Dual cured permanent adhesive resin cement specially formulated for Zirconia cementation.

Ingredients:

- Bis-GMA
- Urethandimethacrylate Oligomer
- Triethyleneglycoldimethacrylate
- Methacrylatedphosphoricacidesters
- 4-Methacryloxyethyltrimelliticacid [3-(Methacryloyloxy) propyltrimethoxysilane
- Photoinitiator
- Co-initiator
- Benzoyl Peroxide
- Bariumaluminoborosilicateglass
- Fumed Silica

Recommended cure times: 20 seconds with a light curing machine

Working steps:

1. Remove the provisional restoration. Note: Use only temporary cements that are eugenol-free. The use of eugenol containing temporary luting materials can inhibit the polymerization process of CyberCyberZirconite
2. Rinse thoroughly.
3. Air dry the preparation (donotdesiccate).
4. Try in the restoration to ensure proper fit.
5. Bleed cartridge before initial use. Place mixing tip on auto-mix syringe.
6. Dispensing the material can be done as follows:
Inlay/Onlay - Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly into the cavity preparation covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all sides.
Crown - Dispense the cement directly into there storation or tooth preparation covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all margins.
Working time: 1-2 minutes - Set Time is 3 minutes (atoraltemperature).
7. When the restoration is properly seated, remove all excesscement. Excess cement is best removed in its gel state with a scaler or explorer.Gel state can be achieved by tack curing excess Ç1.5 minutes before tack curing.
8. After removal of excesscement, light cure all surfaces including margins for 20 seconds.
9. Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.
10. Check occlusion.

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