• Description for:

- **Silk black:** Noncapillary silk has excellent handling and tying characteristics. Modern braiding technique provides a uniform smooth surface and a greater tensile strength.
- **PGA violet:** The extraordinarily smooth, low friction proprietary coating reduces tissue drag. These synthetic absorbable sutures ensure closure through the critical wound healing period. They retain approximately 50% of their high initial strength at two weeks and 20% of their strength at three weeks.
- **PGA rapid:** Developed for short-term closures, is ideal for skin and mucosa closings that may benefit from fast suture absorption. It maintains 50% strength at 5 days, 0% at 14 days and is essentially absorbed at 42 days. It is processed to absorb quickly with little tissue reaction.

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CYBERTECH

SURGICAL SUTURE

- Sharp needles for easy penetration
- Strong needle-to-suture attachment ensures consistent quality and less tissue trauma
- Long cutting edge for suturing control

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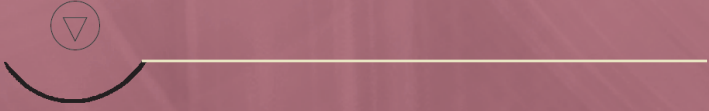
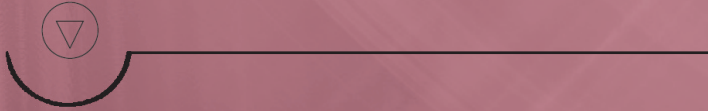
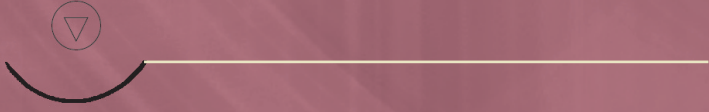
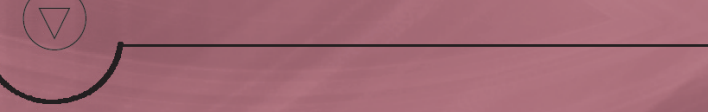
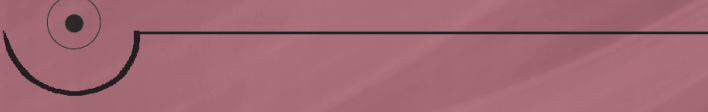
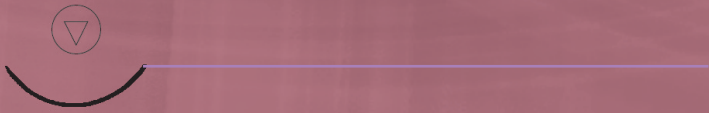
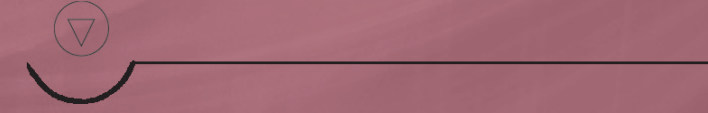
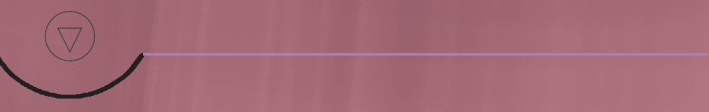
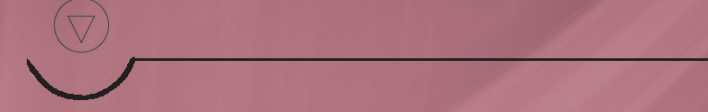
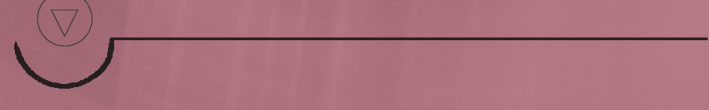
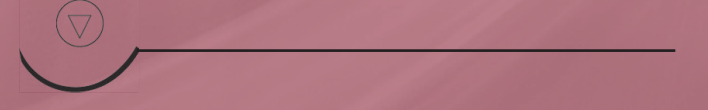
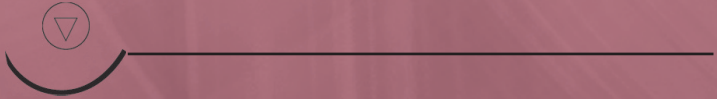
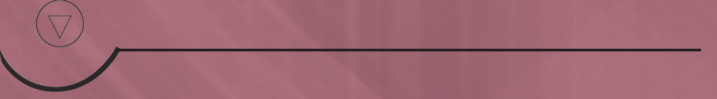
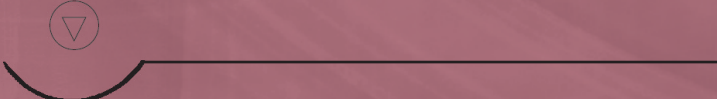
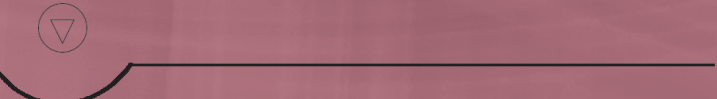
DE Healthcare Ltd
Unit 9, Kingsthorpe Business Centre,
Studland Road, Kingsthorpe,
Northampton NN2 6NE U.K.
www.cybertechbrand.com

SURGICAL SUTURES/SUTURAS QUIRÚRGICAS/SUTURES CHIRURGICALES/CHIRURGISCHES NAHTMATERIAL /SUTURE CHIRURGICHE/CHIRURGISCH

Item Code	Suture material	Needle Code	Length	Indication	Item Code	Suture material	Needle Code	Length	Indication
900 8112	PGA rapid 3-0	3/8RC18 (C-6)	45 cm	<ul style="list-style-type: none"> • Absorbable • Braided • Short term wound support • For superficial soft tissues 	900 8118	Silk black 4-0	1/2 RC20 (C-20)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									
900 8113	PGA rapid 4-0	3/8RC18 (C-6)	45 cm	<ul style="list-style-type: none"> • Absorbable • Braided • Short term wound support • For superficial soft tissues 	900 8119	Silk black 3-0	1/2 RC23 (C-9)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									
900 8114	PGA violet 4-0	1/2TP18 (T-28)	45 cm	<ul style="list-style-type: none"> • Absorbable • Average 50-90 days for absorption 	900 8120	Silk black 3-0	1/2 TP22 (T-14)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									
900 8115	PGA violet 4-0	3/8RC18 (C-6)	70 cm	<ul style="list-style-type: none"> • Absorbable • Average 50-90 days for absorption 	900 8121	Silk black 4-0	3/8PRC13 (C-3P)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									
900 8116	PGA violet 3-0	3/8RC24 (C-7)	70 cm	<ul style="list-style-type: none"> • Absorbable • Average 50-90 days for absorption 	900 8122	Silk black 5-0	3/8RC13 (C-3)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									
900 8117	Silk black 3-0	1/2RC15 (HS-16)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months 	900 8123	Silk black 3-0	3/8RC16 (DS-16)	45 cm	<ul style="list-style-type: none"> • Extremely high surgeons comfort • The closure requires a long-term fixation of less than 6 months
									

SURGICAL SUTURES/SUTURAS QUIRÚRGICAS/SUTURES CHIRURGICALES/CHIRURGISCHES NAHTMATERIAL /SUTURE CHIRURGICHE/CHIRURGISCH

HS Item Code	Suture material	Needle Code	Length	Indication
900 8124	Silk black 4-0	3/8RC16 (DS-16)	45 cm	<ul style="list-style-type: none"> Extremely high surgeons comfort The closure requires a long-term fixation of less than 6 months
				
900 8125	Silk black 5-0	3/8RC16 (DS-16)	45 cm	<ul style="list-style-type: none"> Extremely high surgeons comfort The closure requires a long-term fixation of less than 6 months
				
900 8126	Silk black 3-0	3/8RC18 (C-6)	45 cm	<ul style="list-style-type: none"> Extremely high surgeons comfort The closure requires a long-term fixation of less than 6 months
				
900 8127	Silk black 4-0	3/8RC18 (C-6)	45 cm	<ul style="list-style-type: none"> Extremely high surgeons comfort The closure requires a long-term fixation of less than 6 months
				
900 8128	Silk black 2-0	3/8RC20 (C-20)	45 cm	<ul style="list-style-type: none"> Extremely high surgeons comfort The closure requires a long-term fixation of less than 6 months
				
900 8129	Silk black 3-0	3/8 RC24 (C-7)	45 cm	<ul style="list-style-type: none"> Absorbable • Braided Short term wound support For superficial soft tissues
	